## Opalescence® PF Tooth Whitening System

**Description:** Opalescence® is a dentist-controlled, dentist supervised, take-home bleaching system, dispensed in syringes. It is a clear, fla-vored, high viscosity, sticky, 10% (3.6% HP) and 16% (5.8% HP) carbamide peroxide gel with potassium nitrate and sodium fluoride.

**Bleaching Precautions:**

- Do not use household bleach to whiten teeth.
- Pregnant women should not bleach.
- If you have any questions regarding the appropriate use of this product, including how long it will take to to bleach your teeth, please consult your dentist as soon as possible.
- Do not use tobacco products or eat white bleaching.
- White spots on the enamel or that appear during bleaching may blend during continued bleaching process.
- Teeth are naturally darker along the gumline. These areas may require more time to lighten than the rest of the tooth surface and usually remain slightly darker.

**Precautions for Europe:**

- Not to be used on a person under 18 years of age.
- To be only sold to dental practitioners. For each cycle of use, the first use to be only done by dental practitioners or under their direct supervision if an equivalent level of safety is ensured. Afterwards to be provided to the consumer to complete the cycle of use.
- Contains sodium fluoride.
- Please note the expiration date printed on the outer package/ syringe.

**Precautions:**

- A small percentage of patients experience sensitivity with bleaching. Should this occur, remove the tray and contact your dentist.
- Foods and juices high in citric acid can cause sensitivity to the teeth.
- Some patients have noticed temporary discomfort of the gums, lips, throat, or tongue. Should any of these symptoms persist more than two days or progressively worsen, call your dentist. These side effects will usually subside within 1-3 days after treatment is discontinued.
- Coffee, tobacco, and other products can restrain your teeth over time. Should this occur, the teeth can be rewritten within a few nights using Opalescence.
- Regular dental checkups and cleanings are important before and after bleaching to maintain a healthy smile.
- Some old amalgam or "silver" fillings may leave a dark purple color in your bleaching tray; this is normal.
- Crowns, bridges, partial dentures, veneers, and composite fillings will not bleach.
- Store bleach out of the sun and heat. Refrigeration recommended. Do not freeze.
- Discard any unused bleaching gel after treatment is completed.
- Keep Opalescence out of reach of small children.
- Do not swallow gel or rinsed gel. Product contains peroxide and may contain fluoride; swallowing large amounts can be harmful.
- Materials should be refrigerated (2°–8° C) to maintain shelf life.

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