

Beschreibung des "RECIPROC SYSTEM"

Description of "RECIPROC SYSTEM"

Bei ENDO RADAR PLUS / ENDO RADAR PRO gibt es eine kleine Änderung bei „RECIPROC SYSTEM“, es heißt „Safety Glide Path“ (SGP-System). Bitte überprüfen Sie das Bild.

Sie müssen zuerst dieses SGP-SYSTEM finden und dann TORQUE und SPEED anpassen, um den Feilentyp wie gewünscht einzustellen. Sie können drei Einstellungen speichern.

With ENDO RADAR PLUS/ENDO RADAR PRO there is a little change at "RECIPROC SYSTEM", it is called "Safety Glide Path"(SGP System), please check the picture.

You need to find this SGP SYSTEM firstly, then adjust the TORQUE and SPEED to set the file type as you want, you can store three settings.



Die folgenden Parameter werden häufig für Feilen verwendet.

The following are commonly used FILES parameters. For other models, please consult your supplier.

File system	File No.	Speed	Torque	Forward angle	Reverse angle
WAVE ONE	S	400	4.0	30	150
	P				
	L				
Wave_One_Gold	20/.07	400	4.0	30	150
	25/.07				
	35/.06				
	45/.05				
File system	File No.	Speed	Torque	Forward angle	Reverse angle
RECIPROC	R25	400	4.0	50°	170°
	R40				
	R50				
RECIPROC BLUE	25/.08	400	4.0	50°	170°
	40/.06				
	50/.05				